

## CURLIN STEPS

1. Change batteries of pump and POWER ON
2. Open pump: There is a metal lever on top (says "LIFT TO OPEN"); lift lever then pull open the metal part (pulls open to the right)
3. ATTACH TUBING TO PUMP:
  - a. Find the yellow square; this fits in a hole on the left (there is an arrow pointing to spot)
  - b. Lay tubing along the top to bring blue square to right side
  - c. Blue square fits in a hole on the right (also an arrow point to spot); You may need to pull tubing tight to get blue square in securely
  - d. Close metal "door back over pump and push down lever latch to close
    - i. BEFORE CLOSING, MAKE SURE THE TUBING JUST BELOW THE BLUE SQUARE (WHERE THE DOOR HINGE IS) MOVE OUT OF THE WAY OF THE METAL DOOR OR IT WILL NOT LET IT CLOSE
4. LOOK AT PUMP SCREEN – the word "PROGRAM" is highlighted
  - a. Push YES (green key)
5. Use down arrow to highlight "REPEAT RX"
  - a. Push YES
  - b. The pump screen will start scrolling through the current pump settings – just let it do this and when it gets done, the screen will say "RUN to start" at top of screen
6. PRIME tubing
  - a. Push dark green "PRIME" button at bottom right of pump
  - b. Make sure clamps are open and end cap is off of the end of the tubing
  - c. HOLD DOWN the prime button; pump will start priming; let it get all air out of line (TPN to the end of the line)
  - d. Push YES to exit when done
7. Connect tubing to patient
8. Push the RUN/PAUSE button (top right of pump) to start